


Voltarol



PLEASE TEAR HERE ►

**Heat Patch
Non Medicated**

Tear open the sleeve, when ready to use the heat patch. No shaking of the patch is necessary. Remove the plastic backing then place the heat patch on the painful area, either the back, neck or shoulders, apply the adhesive side onto dry clean skin. Press firmly. The heat patch starts to heat up after a few minutes of opening. When you remove the heat patch, peel it off slowly from one corner.

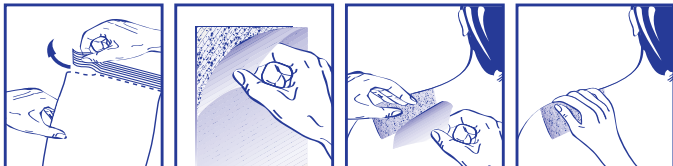

Do not leave the heat patch on the same area of skin for more than 8 hours in any 24 hour period, and do not use more than one heat patch per day on the same area of skin. The heat patch is for external use. Stop use and contact your doctor if after seven days your pain gets worse or remains unchanged. Do not use with other skin medications or other heat sources. Do not attempt to reheat in the oven or microwave. Do not cut, pierce or allow the patch to get wet. For single use.

Do not use if you have previously had an allergic reaction to a heat patch. Do not use on areas of skin where heat cannot be felt, or on damaged skin.

Do not use on children under 12 years of age or persons unable to remove the heat patch on their own. Do not use on bruises or swellings within 48 hours of injury. Do not use if heat patch contents leak and/or wrap is damaged or torn. Do not use while sleeping. Remove product before undergoing any MRI (Magnetic Resonance Imaging) procedures.



Keep out of sight and reach of children.



Haleon Medical Devices

Clocharane, Youghal Road, Dungarvan, Co. Waterford, X35 Y983, Ireland.

UK RP: Haleon UK Trading Limited, The Heights, Weybridge, KT13 0NY, U.K.

Trade marks are owned by or licensed to the Haleon group of companies.

✉ mystery.gb@haleon.com

mystery.ie@haleon.com

☎ **GB: 0800 783 8881**

IE: 1800 441 442

🌐 www.voltarol.co.uk

www.voltarol.ie



See carton

GB 944519
PMF805-V05

CE

1639

MD

3R