

EFFECTIVE STAIN REMOVAL WHITENING OVERNICHT

EVEN WITHOUT BRUSHING

ANTIBACTERIAL DENTAL APPLIANCE CLEANSER

Warnings



Keep tablets out of reach of children and those at risk of accidentally swallowing the tablet



Do not place tablets or solution in mouth



Do not drink cleansing Do not drink cleansing solution or use it as mouthwash

- Do not use if you are sensitive to any of the ingredients.
- Do not place tablets or solution in mouth.
- May produce allergic reaction.
 If irritation occurs, discontinue use and get medical advice and/or attention.
- IF SWALLOWED: Call a Poison Control Center or doctor.
- · Do not touch mouth or eves after handling tablets or
- Wash hands thoroughly after handling tablet or solution.
- · Causes serious eve irritation. IF IN EYES: rinse cautiously with water for several minutes. If eye irritation persists, get medical advice and/or attention.

Inaredients

sodium bicarbonate, citric acid, potassium caroate, sodium carbonate, sodium carbonate peroxide, TAED, sodium lauryl sulfate. PEG-180, sodium benzoate. VP/VA copolymer, flavor, cellulose gum, blue 1 lake, FD&C blue no. 2, yellow 5, yellow 5 lake

Other information

1-866-844-2797

Do not store above 25°C (77°F). Questions or comments?

Tamper evident inner unit

Do not use if sealed Polident packs are torn or broken.

#1 DENTIST RECOMMENDED DENTAL APPLIANCE CLEANSER HALEON

MPROVED

FORMULA




ANTIBACTERIAL DENTAL APPLIANCE CLEANSER



OVERNIGHT

EFFECTIVE STAIN REMOVAL EVEN WITHOUT BRUSHING

Kills 99.9% of odor-causin bacteria+ on:

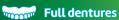



Partial dentures



Retainers & aligners



Mouthguards[^]

60 TABLETS

Why clean daily with POLIDENT?



If not cleaned daily, food particles and build-up can get stuck under your dental appliance, which can lead to unsightly stains and malodor. Removing your dental appliance daily can help relieve soreness and pressure from wearing them across the day.

POLIDENT with more ACTIVE POWER BUBBLES* is a daily cleanser specially designed for dental appliances. It cleans away stains and plaque build-up leaving your dental appliance fresh, all while killing 99.9% of odor-causing

Use daily for a clean, fresh, healthy

- *vs previous POLIDENT range
- +in lab tests
- ^Suitable for common dental appliances

Distributed by: Haleon Warren, NJ 07059

Trademarks are owned by or licensed to the Haleon group of companies. ©2024 Haleon group of companies or its licensor.

Pat. Info www.productpats.com 62000000213151

POLIDENT with more ACTIVE POWER BUBBLES* for an amazingly clean dental appliance, every day!

POLIDENT with more ACTIVE POWER BUBBLES* is a daily cleanser that actively cleans even the hardest to reach places⁺ of your dental appliance. Each tablet is made with a specially designed combination of ingredients generating more ACTIVE POWER BUBBLES* that are tough on dirt but gentle on your dental appliance - removing stains and plaque build-up - leaving it clean and fresh. Just soak, brush, rinse, and go!



IMPORTANT: Read all directions before use.



DIRECTIONS FOR DAILY USE

1. Drop one POLIDENT cleanser tablet into enough very warm (not hot) water.



2. Add denture or dental appliance to the soaking solution. Ensure that it is fully covered.



3. After soaking overnight, brush denture or dental appliance with solution using a soft brush. Discard remaining solution immediately after use. Rinse thoroughly under running water.

