DataMatrix: 00213152



SMOKERS EFFECTIVELY CLEANS TOUGH STAINS LIKE COFFEE, TEA & TOBACCO

ANTIBACTERIAL DENTAL APPLIANCE CLEAN

Warnings

Keep tablets out of reach of children and those at risk of accidentally swallowing the tablet or solution



Do not drink cleansing solution or use it as mouthwash

• Do not use if you are sensitive to any of the ingredients.

in mouth

- Do not place tablets or solution in mouth.
- May produce allergic reaction. If irritation occurs, discontinue use and get medical advice and/or attention.
- IF SWALLOWED: Call a Poison Control Center or doctor.
- Do not touch mouth or eyes after handling tablets or solution.
- Wash hands thoroughly after handling tablet or solution.
- Causes serious eye irritation. IF IN EYES: rinse cautiously with water for several minutes. If eye irritation persists, get medical advice and/or attention.

Ingredients

sodium bicarbonate, citric acid, potassium caroate, sodium carbonate, sodium carbonate peroxide, TAED, sodium lauryl sulfate, PEG-180, sodium benzoate, VP/VA copolymer, flavor, cellulose gum, blue 1 lake, FD&C blue no. 2, yellow 5, yellow 5 lake

Other information

Do not store above 25°C (77)

Do not store above 25°C (77°F).

Questions or comments? 1–866–844–2797

Tamper evident inner unit

Do not use if sealed Polident packs are torn or broken.

#1 DENTIST RECOMMENDED HAL≣ON DENTAL APPLIANCE CLEANSER POLIDENT MPROVED **ANTIBACTERIAL DENTAL APPLIANCE CLEANSER** FORMULA **SMOKERS ACTIVE** POWER **BUBBLES EFFECTIVELY CLEANS TOUGH STAINS** LIKE COFFEE, TEA & TOBACCO Kills 99.9% of odor-causing bacteria+ on: POLIDENT Full dentures Partial dentures Retainers & aligners^{^1} 90 TABLETS Mouthguards[^]

Why clean daily with POLIDENT?



If not cleaned daily, food particles and build-up can get stuck under your dental appliance, which can lead to unsightly stains and malodor. Removing your dental appliance daily can help relieve soreness and pressure from wearing them across the day.

POLIDENT with more ACTIVE POWER BUBBLES* is a daily cleanser specially designed for dental appliances. It cleans away stains and plaque build-up leaving your dental appliance fresh, all while killing 99.9% of odor-causing bacteria*.

Use daily for a clean, fresh, healthy smile!

*vs previous POLIDENT range tin lab tests

Pat. Info www.productpats.com

^Suitable for common dental appliances

Distributed by: Haleon Warren, NJ 07059

Trademarks are owned by or licensed to the Haleon group of companies. ©2024 Haleon group of companies or its licensor.

62000000213152

POLIDENT with more ACTIVE POWER BUBBLES* for an amazingly clean dental appliance, every day!

POLIDENT with more ACTIVE POWER BUBBLES* is a daily cleanser that actively cleans even the hardest to reach places* of your dental appliance. Each tablet is made with a specially designed combination of ingredients generating more ACTIVE POWER BUBBLES* that are tough on dirt but gentle on your dental appliance – removing stains and plaque build-up – leaving it clean and fresh. Just soak, brush, rinse, and go!





1. Drop one POLIDENT cleanser tablet into enough very warm (not hot) water.



2. Add denture or dental appliance to the soaking solution. Ensure that it is fully covered.



 After soaking for 15 minutes, brush denture or dental appliance with solution using a soft brush. Discard remaining solution immediately after use. Rinse thoroughly under running water.

