


Int. 2 of 5 : 0000219386

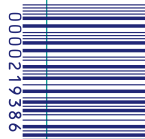

0000219386

963612



3 10158 08025 1

Questions or comments? 1-866-844-2797


Helpful Hint:
Another way to apply is to place
dry strips on a **wet** denture.



Comfort Seal
Strips
for Uppers, Lower & Partial

40 STRIPS

Comfort Seal
Strips
for Uppers, Lower & Partial

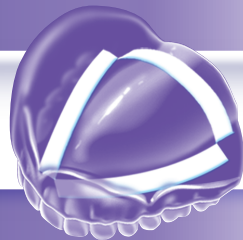

- *Helps Seal Out Food Particles*
- *No Messy Ooze**
- *Zinc-Free*

40
STRIPS

Ingredients: PEG-90M, microcrystalline wax, polybutene, cellulose gum
Trademarks are owned by or licensed to the Haleon group of companies.
©2025 Haleon group of companies or its licensor.
Distributed by: Haleon, Warren, NJ 07059
Made in Japan

219386

HALEON

SUPER STRONG, ALL-DAY HOLD

POLIGRIP
DENTURE ADHESIVE STRIP

Ill-fitting dentures may impair your health.
Visit your dentist regularly to ensure proper
fitting dentures. Do not use if you have sensitivity
to or reaction from any of the strip ingredients.
CAUTION: Keep out of reach of children.
Do not store above 30°C (86°F).

Upon application

EXP:

LOT:

To remove dentures and adhesive:

1. Swish your mouth with warm water and take denture out slowly by using a rocking motion.
2. Wipe adhesive residue from gums and denture with a soft, moist cloth.
3. Brush denture with a soft brush under running water and soak in Polident Denture Cleanser.



1-866-844-2797



1. Clean and dry your denture.



2. With dry hands, peel open one strip at a time with water.



3. Lightly moisten one strip edges. Do not overlap strips.



4. Place strips as shown, not too close to denture edges. Do not overlap strips.



5. Rinse mouth with water before inserting denture.

6. Press denture in place, hold firmly and bite down.

ID# INSIDE