

UPPER



LOWER



PARTIAL



**Helpful Hints:** For the first time, use a SMALL amount. Use more if needed. Too much adhesive can cause oozing; apply less next time if this occurs. It may take a few tries to find the right amount for your denture. Apply once a day for secure hold. **If you need to apply more than once a day, you may have an ill-fitting denture and should seek the advice of your dental care professional.** To keep tip from clogging, keep cap and nozzle dry. Replace the cap after use. **KEEP CARTON FOR COMPLETE DIRECTIONS AND WARNINGS.**

### Directions

#### APPLYING:

1. Clean and dry your dentures.
2. Apply Poligrip in SHORT STRIPS, as shown in the diagram, not too close to the denture edges.
3. Rinse mouth before inserting dentures.
4. Press dentures into place, hold firmly, and bite down for a few seconds to secure hold.

#### REMOVING:

1. Swish mouth with water.
2. Slowly remove denture using a rocking motion.
3. Remove adhesive residue from denture and mouth with warm water and a soft brush.
4. Use POLIDENT Denture Cleanser to thoroughly clean your denture and then rinse with water.



**IMPORTANT**  
READ INSTRUCTIONS FIRST

LOT



ZINC FREE

**SUPER** STRONG, ALL-DAY HOLD  
**POLIGRIP**  
DENTURE ADHESIVE CREAM

NET WT.  
2.2 OZ (62 g)

HALEON

HELPS SEAL OUT IRRITATING FOOD PARTICLES

**EXTRA CARE**

TRIPLE ACTION: STRONG HOLD, SEAL &amp; COMFORT

6200000219607

ZINC FREE

**SUPER** STRONG, ALL-DAY HOLD  
**POLIGRIP**  
DENTURE ADHESIVE CREAM

NET WT.  
2.2 OZ (62 g)

HALEON

HELPS SEAL OUT IRRITATING FOOD PARTICLES

**EXTRA CARE**

TRIPLE ACTION: STRONG HOLD, SEAL &amp; COMFORT

### ✓ Help to seal out food particles and prevent gum irritation

Poligrip Denture Adhesive Cream is clinically proven to increase the chewing performance of your denture. Not only does it provide all day hold, it boosts your chewing power when you eat. We know food matters, so now you can dig into the things you crave with confidence. Enjoy more chewing power with Poligrip\*.

\*Chewing performance measured in a clinical study vs. no adhesive.

### Warnings

A few individuals may be sensitive or allergic to this product. If you experience an allergic reaction or discomfort, discontinue use immediately. Swallowing small amounts of this product, when used as directed, may occur and is not harmful. Other side effects such as oral irritations and gastrointestinal symptoms (e.g. nausea) may occur.

Keep out of sight and reach of children. Ill-fitting dentures may impair your health. Visit your dentist regularly to check your denture fit.

### Ingredients

calcium/sodium PVM/MA copolymer, petrolatum, cellulose gum, mineral oil, tocopheryl acetate

### Other information

Do not store above 30°C (86°F).



Trademarks are owned by or licensed to the Haleon group of companies. ©2025 Haleon group of companies or its licensor.  
Distributed by: Haleon  
Warren, NJ 07059  
Made in Ireland  
6200000219607  
Questions or comments?  
1-866-844-2797