



LOT

Powder

Extra Strength

POLIGRIP
 SUPER STRONG, ALL-DAY HOLD
 DENTURE ADHESIVE POWDER
Directions:

For the first time, spray very lightly. Use more if needed. Too much adhesive can cause oozing. It may take a few tries to find the right amount for your denture. Apply once a day for secure hold. If you need to apply more than once a day, seek advice from your dental care professional.

APPLYING

1. Clean dentures thoroughly and leave wet.
2. Spray powder very lightly and evenly onto denture surfaces coming into contact with gums or roof of mouth.
3. Thoroughly shake off any excess powder.
4. Press dentures into place, hold firmly, and bite down for a few seconds to secure hold.

FOR UPPERS

Position bottle as illustrated below so powder is dispensed over a large surface. Move tip from front to back of denture.

SIDE

**FOR LOWERS**

Position bottle as illustrated below so powder is dispensed in thin stream. Move tip along "U" of denture channel.

FRONT

**REMOVING**

1. Swish mouth with water.
2. Slowly remove denture using a rocking motion.
3. Wipe adhesive residue from denture and mouth with soft, moist cloth.
4. Use Polident Cleanser to thoroughly clean your denture and then rinse with water.

Warnings:

Keep out of sight and reach of children.

Swallowing small amounts of these products, when used as directed, may occur and is not harmful.

Ill-fitting dentures may impair your health. Visit your dentist regularly to check your denture fit.

To avoid clogging, keep dispenser tip dry. Replace the cap after use. A few individuals may be sensitive or allergic to this product. If you experience an allergic reaction or discomfort, discontinue use immediately. Other side effects such as oral irritations and gastrointestinal symptoms (e.g. nausea) may occur.

1-866-640-1017

POLIGRIP
 SUPER STRONG, ALL-DAY HOLD
 DENTURE ADHESIVE POWDER


62000000077314

POLIGRIP
 SUPER STRONG, ALL-DAY HOLD
 DENTURE ADHESIVE POWDER

Extra Strength

Powder

Super Poligrip Extra Strength Powder provides strong hold in a convenient powder form. It is zinc free and works all day to help seal out food particles thus helping to protect gums from irritation.

Distributed by:
GSK Consumer Healthcare
 Warren, NJ 07059
 Made in Ireland

Trademarks are owned by or licensed to the GSK group of companies.

Questions or comments?
 1-866-640-1017

©2022 GSK group of companies or its licensor.

gsk

SUPER STRONG, ALL-DAY HOLD

POLIGRIP
 DENTURE ADHESIVE POWDER

Extra Strength

Powder

- 12-Hour formula
- Original fresh flavor
- Helps seal out food particles

NET WT. 1.6 OZ (45 g)



3 10158-07801 2



Powder

Extra Strength

POLIGRIP
 SUPER STRONG, ALL-DAY HOLD
 DENTURE ADHESIVE POWDER
Ingredients:

calcium/sodium PVM/MA copolymer, cellulose gum, flavor

- Get the confidence you need with the strong, all-day hold of SUPER POLIGRIP EXTRA STRENGTH POWDER.**
- Helps hold dentures tight for up to 12 full hours.
 - Makes your dentures feel comfortable and natural in your mouth.
 - Forms a seal to help keep food from getting stuck between your dentures and gums.
 - Has an easy-to-use, convenient bottle.