

UPPER



LOWER



PARTIAL



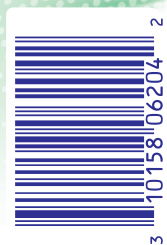
Helpful Hints: For the first time, use a SMALL amount. Use more if needed. Too much adhesive can cause oozing; apply less next time if this occurs. It may take a few tries to find the right amount for your denture. Apply once a day for secure hold. **If you need to apply more than once a day, you may have an ill-fitting denture and should seek the advice of your dental care professional.** To keep tip from clogging, keep cap and nozzle dry. Replace the cap after use. **KEEP CARTON FOR COMPLETE DIRECTIONS AND WARNINGS.**

APPLYING:

1. Clean and dry your dentures.
2. Apply Poligrip Free in **SHORT STRIPS**, as shown in the diagram, not too close to the denture edges.
3. Rinse mouth before inserting dentures.
4. Press dentures into place, hold firmly, and bite down for a few seconds to secure hold.

REMOVING:

1. Swish mouth with water.
2. Slowly remove denture using a rocking motion.
3. Remove adhesive residue from denture and mouth with warm water and a soft brush.
4. Use POLIDENT Denture Cleanser to thoroughly clean your denture and then rinse with water.

**IMPORTANT**

READ INSTRUCTIONS FIRST

LOT



ZINC FREE

SUPER STRONG, ALL-DAY HOLD
POLIGRIP
DENTURE ADHESIVE CREAM

NET WT.
2.4 OZ (68 g)

HALEON

NO ARTIFICIAL COLORS OR FLAVORS

FREE

TRIPLE ACTION: STRONG HOLD, SEAL & COMFORT

62000000219367

ZINC FREE

SUPER STRONG, ALL-DAY HOLD
POLIGRIP
DENTURE ADHESIVE CREAM

NET WT.
2.4 OZ (68 g)

HALEON

NO ARTIFICIAL COLORS OR FLAVORS

FREE

TRIPLE ACTION: STRONG HOLD, SEAL & COMFORT

✓ Free of artificial colors & flavors

Poligrip Denture Adhesive Cream improves the performance of your denture. Not only does it provide all day hold, it boosts your chewing power* when you eat. We know food matters, so now you can dig into the things you crave with confidence. Enjoy more chewing power with Poligrip*.

*Chewing performance measured in a clinical study vs. no adhesive.

INGREDIENTS: calcium/sodium PVM/MA copolymer, petrolatum, cellulose gum, mineral oil

A few individuals may be sensitive or allergic to this product. If you experience an allergic reaction or discomfort, discontinue use immediately. Swallowing small amounts of this product, when used as directed, may occur and is not harmful.

Other side effects such as oral irritations and gastrointestinal symptoms (e.g. nausea) may occur.

Keep out of sight and reach of children.

Ill-fitting dentures may impair your health. Visit your dentist regularly to check your denture fit.

Store below 30°C (86°F).



Trademarks are owned by or licensed to the Haleon group of companies. ©2025 Haleon group of companies or its licensor.

Distributed by: Haleon,
Warren, NJ 07059
Made in Ireland
62000000219367

Questions or comments?
1-866-844-2797