

PARTIAL

Helpful Hints: For the first time, use a SMALL amount. Use more if needed. Too much adhesive can cause oozing; apply less next time if this occurs. It may take a few tries to find the right amount for your denture. Apply once a day for secure hold. If you need to apply more than once a day, seek the advice of your dental care professional. To keep tip from clogging, keep cap and nozzle dry. Replace the cap after use. KEEP CARTON FOR COMPLETE DIRECTIONS AND WARNINGS.

APPLYING:

secure hold.

- 1. Clean and dry your dentures.
- 2. Apply Poligrip as shown in the diagram, not too close to the denture edges.
- a. Upper denture: one long continuous strip with 2 shorter strips in the middle
- b. Lower denture: one long continuous strip
- c. Partial denture: 2 short strips3. Rinse mouth before inserting dentures.
- Press dentures into place, hold firmly, and bite down for a few seconds to

REMOVING:

Swish mouth with water.
 Slowly remove denture using a rocking motion.

- Remove adhesive residue from denture and mouth with warm water and a soft bristle brush.
- Use a denture cleanser, such as POLIDENT to thoroughly clean your denture and then rinse with water.





HAL≣ON

POLIGRIP POWER MAX DENTURE ADHESIVE CREAM

12h

POWER
HOLD-COMFORT

HAL≣ON



POLIGRIP POWER MAX DENTURE ADHESIVE CREAM

NET WT 2.2 OZ (62.4 g)

NET WT 2.2 OZ (62.4 g)

12_h

POWER
HOLD +
COMFORT

FLAVOR FREE



New Poligrip Power Hold + Comfort is designed for comfort and to secure your partial and full dentures in place all day. It provides a soft cushioning layer to help absorb the pressure from your partial and full dentures. The precision nozzle allows you to apply Poligrip with control and precision.

INGREDIENTS: cellulose gum, petrolatum, mineral oil, calcium/sodium PVM/MA copolymer, polyacrylic acid

A few individuals may be sensitive or allergic to this product. If you experience an allergic reaction or discomfort, discontinue use immediately. Swallowing small amounts of this product, when used as directed, may occur and is not harmful.

Other side effects such as oral irritations and gastrointestinal symptoms (e.g., nausea) may occur.

Keep out of sight and reach of children.

Ill-fitting dentures may impair your health. Visit your dentist regularly to check your denture fit.

Do not store above 30°C (86°F).

Questions or comments? 1–866–844–2796

Do not use if the sealed carton is broken or damaged.

Pat. Info. www.productpats.com





Distributed by: Haleon, Warren, NJ 07059
Made in Ireland
Trademarks are owned by or licensed to
the Haleon group of companies.
©2024 Haleon group of companies or its
licensor.

*vs. no adhesive

62000000216335