

Helpful Hints:

NET WT

2.2 OZ (62.4 g)

For the first time, use a SMALL amount. Use more if needed, Too much adhesive can cause oozing: apply less next time if this occurs. It may take a few tries to find the right amount for your denture. Apply once a day for secure hold. If you need to apply more than once a day, seek the advice of your dental care professional. To keep tip from clogging, keep cap and nozzle dry. Replace the cap after use. KEEP CARTON FOR COMPLETE DIRECTIONS AND WARNINGS.

APPLYING:

- 1. Clean and dry your dentures.
- 2. Apply Poligrip as shown in the diagram, not too close to the denture edges.
- a. Upper denture: one long continuous strip with 2 shorter strips in the middle b. Lower denture: one long continuous strip c. Partial denture: 2 short strips
- 3. Rinse mouth before inserting dentures.
- 4. Press dentures into place, hold firmly, and bite down for a few seconds to secure hold.

REMOVING:

- 1. Swish mouth with water.
- 2. Slowly remove denture using a rocking motion.
- 3. Remove adhesive residue from denture and mouth with warm water and a soft bristle brush.
- 4. Use a denture cleanser, such as POLIDENT to thoroughly clean your denture and then rinse with water.





POLIGRIP DENTURE ADHESIVE CREAM

POWER HOLD+ 5X FOOD BLOCK*

HALEON

HALEON



POLIGRIP DENTURE ADHESIVE CREAM

NET WT 2.2 OZ (62.4 g)

POWER HOLD+

5X FOOD BLOCK*

Poligrip Power Hold + Seal delivers up to a 12-hour hold* and food seal protection. The precision nozzle enables exactly the right amount of product to be applied around the edge of your denture forming an all-around seal.

*Vs no Adhesive

INGREDIENTS: calcium/sodium PVM/MA copolymer, petrolatum, cellulose gum, mineral oil

A few individuals may be sensitive or allergic to this product. If you experience an allergic reaction or discomfort, discontinue use immediately. Swallowing small amounts of this product, when used as directed, may occur and is not harmful.

Other side effects such as oral irritations and gastrointestinal symptoms (e.g. nausea) may occur. Keep out of sight and reach of children.

Ill-fitting dentures may impair your health. Visit your dentist regularly to check your denture fit.



Recycled

1-866-844-2797 **✓ FSA ELIGIBLE**

Carton

Store below 30°C (86°F). Trademarks are owned by or licensed to the Haleon group of companies.

Questions or comments?

©2025 Haleon group of companies or its licensor. Made in Ireland

Distributed by: Haleon, Warren, NJ 07059 62000000219587





62000000219587