

UPPER



LOWER



PARTIAL

**Helpful Hints:**

For the first time, use a SMALL amount. Use more if needed. Too much adhesive can cause oozing; apply less next time if this occurs. It may take a few tries to find the right amount for your denture. Apply once a day for secure hold. **If you need to apply more than once a day, seek the advice of your dental care professional.** To keep tip from clogging, keep cap and nozzle dry. Replace the cap after use. **KEEP CARTON FOR COMPLETE DIRECTIONS AND WARNINGS.**

APPLYING:

1. Clean and dry your dentures.
2. Apply Poligrip as shown in the diagram, not too close to the denture edges.
 - a. Upper denture: one long continuous strip with 2 shorter strips in the middle
 - b. Lower denture: one long continuous strip
 - c. Partial denture: 2 short strips
3. Rinse mouth before inserting dentures.
4. Press dentures into place, hold firmly, and bite down for a few seconds to secure hold.

REMOVING:

1. Swish mouth with water.
2. Slowly remove denture using a rocking motion.
3. Remove adhesive residue from denture and mouth with warm water and a soft bristle brush.
4. Use a denture cleanser, such as POLIDENT to thoroughly clean your denture and then rinse with water.



ULTIMATE

LOT



NET WT
2.2 OZ (62.4 g)

POLIGRIP
POWER MAX
DENTURE ADHESIVE CREAM

OUR BEST BITING POWER*

ULTIMATE

✓ 12h STRONG HOLD ✓ SEAL
✓ COMFORT ✓ FLAVOR FREE

ZINC FREE

HALEON



62000000219109

NET WT
2.2 OZ (62.4 g)

POLIGRIP
POWER MAX
DENTURE ADHESIVE CREAM

OUR BEST BITING POWER*

ULTIMATE

✓ 12h STRONG HOLD ✓ SEAL
✓ COMFORT ✓ FLAVOR FREE

ZINC FREE

HALEON



- ✓ Poligrip's best biting power technology
- ✓ Precision nozzle
- ✓ Feel no food after eating**

INGREDIENTS: calcium/sodium PVM/MA copolymer, petrolatum, cellulose gum, mineral oil

*Includes Poligrip's best biting power technology.
**75% of people agreed (n=48).

A few individuals may be sensitive or allergic to this product. If you experience an allergic reaction or discomfort, discontinue use immediately. Swallowing small amounts of this product, when used as directed, may occur and is not harmful.

Other side effects such as oral irritations and gastrointestinal symptoms (e.g., nausea) may occur. Keep out of sight and reach of children.

Ill-fitting dentures may impair your health. Visit your dentist regularly to check your denture fit.
Do not store above 30°C (86°F).



Questions or comments?
1-866-844-2797

www.polident.com

Distributed by:
Haleon
Warren, NJ 07059

Trademarks owned or
licensed by Haleon.

Made in Ireland

©2025 Haleon or licensor.
62000000219109

