Order – AWF	AWF 28971 – OTDC 480722
Reference	OTRIVIN SPRAY SOLUTION SEA WATER 50-100 ML HK_941021_PIL
Format	135 x 200 mm – Recto/Verso
Pharma code	–
Colour	P Reflex Blue U
Font	Helvetica Neue LT Pro – Hiragino Kaku Gothic ProN – Text outlined
Font min. size	Text body: 8 pts
Reference MKT	–
Production site	SPPH – FR
Reference SITE	ANA90NC02B
Pre-press	Diadeis Suisse SA – AL – DO-18-E0299 – 14.03.2018 – F27

Recto
135 mm

200 mm

Otrivin® Seawater isotonic solution for daily nasal cleaning.

Properties:
Otrivin® Comfort Seawater Spray is a sea water isotonic solution which is microbiologically pure, without preservative and naturally rich in sea microelements. It moisturizes the nasal mucosa when it is dry and crusty, washes away nasal secretions in case of nasal colds and blocked nose, aids comfortable breathing. Suitable for use in adults and children* for daily nasal cleaning.

Indication

- Washing of nasal cavities.
Washes away nasal secretions in case of nasal colds and blocked nose, aids comfortable breathing.
- Moisturizing of the nasal mucosa.
Moisturizes to keep the mucosa in very good hydrating condition. Improves the drying condition due to temperature change.
- Protecting of nasal mucosa from the irritation.
Protects the mucosa from irritating action of air pollution & sources of allergy.

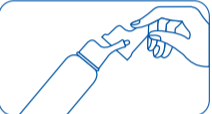
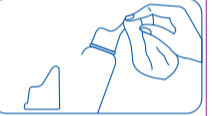
**Abundant mucus production is very frequent in infancy: nasal obstruction, rhinorrhea, but also coughing and vomiting are the most common signs. In these situations child has difficulty in breathing, is irritable and restless, has problems to eat and sleep. Moreover, if the problem is neglected, more serious complications such as sinusitis and otitis may arise.*

Components:
100 ml of solution contain: sea water 31.82 ml purified water up to 100 ml
Inert propellant gas: nitrogen
(Nitrogen is not a component of the solution but acts only as propeller and does not exit the bomb).
It contains no preservatives or medical ingredient.

Frequency of use:
Otrivin® Seawater may be used at any time of the day and in any position. Washing of nasal cavity associated with nasal colds and allergy symptoms: as needed at various time a day. For daily nasal cleaning: 1-2 times a day or more as needed.

Instructions for use:

1. Remove the protective cap.
2. Place the nozzle at the base of the nostril.
3. Exert a light pressure at the nozzle's base*.
4. Repeat the procedure in the other nostril*.
5. After use, clean and dry the nozzle and put the protective cap back on.

* press for 1 sec or longer as needed

Warning:
Contents under pressure: keep away from direct sunlight and do not expose to temperatures over 50°C. Do not puncture or burn it, even after use. Keep out of the reach of children. Refrain from use in case of hypersensitivity to any of the product's components. Use the product exclusively for the application it was intended. Do not use product after its expiry date. The expiry date refers to the product when properly stored.

CE 0459

Information updated: August 2016

安鼻靈 and Otrivin are trade marks owned by or licensed to the GSK group of companies.
© 2016 GSK group of companies or its licensor. All rights reserved.

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